

SNS56712-2"x2"  
SNS56744-4"x4"  
SNS56716-4"x4.75"

## Instructions for Use

### Fibergel Ag Wound Dressing

Simpurity™ Fibergel Ag Dressing is a hydrofiber material derived from calcium alginate, carboxymethylcellulose and a silver compound. It is designed to be highly absorbent. The dressing absorbs exudate and forms a gel-like covering over the wound, maintaining a moist environment for wound healing. The gel-like surface also keeps the dressing from adhering to the wound for painless removal. Simpurity™ Fibergel Ag has sustained silver ion release allowing antimicrobial effects for up to 7 days.

#### **Application:**

1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
2. Choose the right size of dressing to cover the wound according to wound area.  
Use a sterile pair of scissors to cut the dressing or fold it to fit into the wound bed
3. Apply the dressing to the wound (either side of this dressing can be applied to the wound bed)
4. Cover and secure the dressing with a secondary dressing

#### **Changing Frequency:**

Dressing change frequency will depend on the patient's condition and the level of exudate. This dressing is recommended to be changed daily.

#### **Dressing Removal:**

Gently remove the secondary dressing. Repeat application steps.

#### **Indications:**

Simpurity™ Fibergel Ag Wound Dressings are intended for infected moderate to heavily exuding wounds that are partial to full thickness including: trauma wounds, pressure ulcers, diabetic ulcers, leg ulcers, and post-operative surgical wounds.

#### **Contraindications:**

Do not use Simpurity™ Fibergel Ag Dressing on patients with known sensitivity to silver. Not intended for use on third-degree burns.