

Instructions for Safe n Simple **Peri-Stoma Cleanser & Adhesive Remover**

These large soft wipes remove oil, dirt and adhesive residue from the skin so the ostomy pouch will adhere well to the skin.



General Instructions for use:

1. Peel back the label slowly
2. Remove wipe
3. Reseal package to keep wipes moist
4. Gently wipe skin clean
5. No need to rinse skin
6. Make sure skin is dry before going on to using a skin barrier
7. Used wipe is flushable (if not on a septic system or an older plumbing system)

Helps to Remove Pouches:

1. Gently lift a corner or edge of the wafer, barrier, tape or hydrocolloid dressing.
2. Use the wipe between the skin and the dressing, gently peeling the adhesive back while continuing to wipe.
3. Repeat steps 4-7 above.

NOTE: More than one wipe may be needed, depending on how much adhesive is on the skin.

CAUTION: FOR EXTERNAL USE ONLY

Ingredients: Water, Polysorbate 20, Dialzolidoinyl urea, Disodium cocoamphodiacetate, Citric Acid, Iodopropynyl Butylcarbamate.

Safe n Simple™
Clean. Comfortable. Confident.

Clarkston, MI 48348 · P: 844.767.6334 · F: 248.575.4172

www.SnS-Medical.com