

Instructions for Safe n Simple Integrity Skin Barrier Ring

Creates a secure and comfortable seal around the stoma to protect the skin from leakage and irritation. Ideal for a Colostomy which does not need the absorption.



Directions for Application:

Make sure the skin surrounding the stoma is clean, dry and free from oils, creams, and/or soap residues

1. Remove the release liners from both sides of the seal

Integrity seal can be used in two ways:

1. Apply directly on skin: Use your fingers to stretch and shape the seal to fit around your stoma. Make sure you have a snug or comfortable fit...or

2. Apply directly on wafer/skin barrier: Use your fingers to stretch and shape the seal to fit the desired shape around the hole of your wafer/skin barrier pouching system. Center carefully and press firmly in place

Directions for Removal:

1. Gently peel away from skin, working downward and pressing against the surrounding skin

NOTE: Integrity seals can be remolded into any shape and used as caulk to fill in uneven skin areas