

## Instructions for Safe n Simple Skin Barrier Ring - LONGEVITY Conforming Seal

Creates a secure and comfortable seal around the stoma to protect the skin from leakage and irritation. Ideal for all types of ostomies.



### **HOW TO USE LONGEVITY SKIN BARRIER RINGS**

Longevity rings have two unique layers that function differently when applied to the skin.

**WHITE LAYER AGAINST SKIN:** for stomas with thin to liquid output (ileostomy, urostomy) the ring provides a secure seal at the skin surface with minimal if any moisture absorption.

**TAN LAYER AGAINST SKIN:** for stomas with thick to solid output (colostomy) the ring maintains a secure seal at the skin surface but can absorb moisture when needed.

### **SKIN PREPARATION AND RING APPLICATION**

Make sure the skin surrounding the stoma is clean, dry and free from oils, creams and/or soap residues. Remove label release papers from ring and shape or stretch to fit your stoma or around the cut opening on the back of your pouch or wafer. Make sure you have a snug fit around the stoma or wafer opening and apply to skin. Press firmly in place to mold the seal and a comfortable fit.

**Composition:** Hydrocolloid and Zinc Oxide



Clarkston, MI 48348 • P: 844.767.6334 • F: 248.575.4172  
samples@sns-medical.com • www.sns-medical.com

Like us on Facebook: <https://www.facebook.com/Safe-n-Simple>